



Filters for embryo collection



User-friendly and safe, requiring no monitoring during flushing procedure


- + All-in-one dual-purpose filters and search dishes, embryos are recovered directly from the filter dish
- + Also suitable for filtering oocytes after OPU
- + Secure fixation of the lid during collection and easy to open after collection (no screw cap)
- + Integrated search dish engraved with a grid makes it easy to search and find embryos (mesh size is ideal for all bovine and equine embryo sizes)
- + Filter mesh cannot get dry as it remains in contact with fluid
- + Excess fluid is controlled and removed through outlet tubing with attached thumb clamp (entry hole is big enough to ensure the necessary flow speed of the flushing medium)
- + Connection to all types of Y-tubing (bovine and equine) is possible
- + Sterilized and absolutely tight

EmSafe

The EmSafe is ideal for both large and small embryos. The filter system automatically holds a minimum level of 60 ml flushing fluid in the dish. The angled filter mesh provides an optimal flow-through rate and the air vent in the lid is secured by a filter. The air outlet is also provided with a filter to securely hold embryos in the system.

EmSafe filter [1]

19010/6000

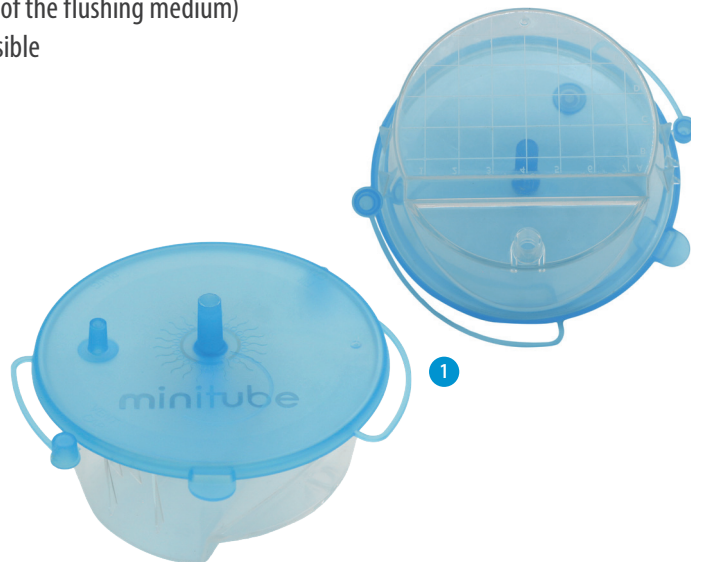

Accessories

Pouch for EmSafe filter [2]

19010/6050

Y-junction tubing for EmSafe filter, sterilized, 1/bag

Luer [3]

19011/0100

Foley [4]

19011/0200



minitube



Miniflush®

Vertical all-in-one embryo recovery system with a large filter surface area. The vertical design provides higher flow rates and reduces plugging of the embryo filter. Due to the shallow design, the filter is perfectly suited for use with all microscopes even when the working distance is limited. Embryos are easily found with the aid of the highly visible grid. With the Miniflush®, the entire flushing procedure can be completed without the need to monitor fluid levels as the filter automatically maintains a constant 20 ml volume.

Miniflush® filter [19222/2000](#)

Accessories

Pouch for Miniflush® filter [2] [19010/1100](#)

Y-junction tubing for Miniflush® filter, sterilized, 1/bag

Luer [3] [19011/0000](#)

Foley [4] [19011/0001](#)

