



# AI TRAINING MODEL "SUZY"



## A new approach to artificial insemination training

"Suzy" is an artificial sow for training AI technicians in a safe and convenient environment, before they work with real animals.

The training model consists of a plastic body and silicone internals. It provides a most realistic gynaecological tract (GT) that can be removed to simplify training in the initial stages. "Suzy's" GT features a cervix, similar to the natural one, for practising the lock of the AI catheter in the cervix. A sound will confirm success when the introduced catheter reaches the correct position in the cervix. A replica of the urethra opening trains the correct introduction of the AI catheter avoiding positioning it in the urethra. Inseminating "Suzy" is also possible, as the GT has a drain.

The artificial sow provides flexibility around timing and location of AI training courses, and also lowers the risk of injury to animals by unexperienced trainees. Additionally, training units can be repeated as often as necessary. All these benefits help to provide optimal training and ultimately an improved performance of applicants of AI in sows.

### Product features

- Realistic sow model, gynaecological tract and cervix
- Confirmation sound when the AI catheter has been introduced correctly
- Trains correct introduction of the AI catheter avoiding positioning it in the urethra
- Training of actual insemination process possible
- Compact and light-weight training model (e.g. it fits in the boot of a station wagon or similar car size)

### Artificial sow "Suzy" with complete gynaecological tract

and starter-kit including 100 AI catheters and two tubes of ReproJelly

REF. : [22400/0100](#)

### Spare parts: Genital tract for "Suzy"

Part 1: Outer part with vulva and tail

REF. : [22400/0105](#)

Part 2: Cervix and Oinkmaster

REF. : [22400/0106](#)

Correct catheter introduction on removed GT - prevent mistaken introduction of AI catheter in urethra

