


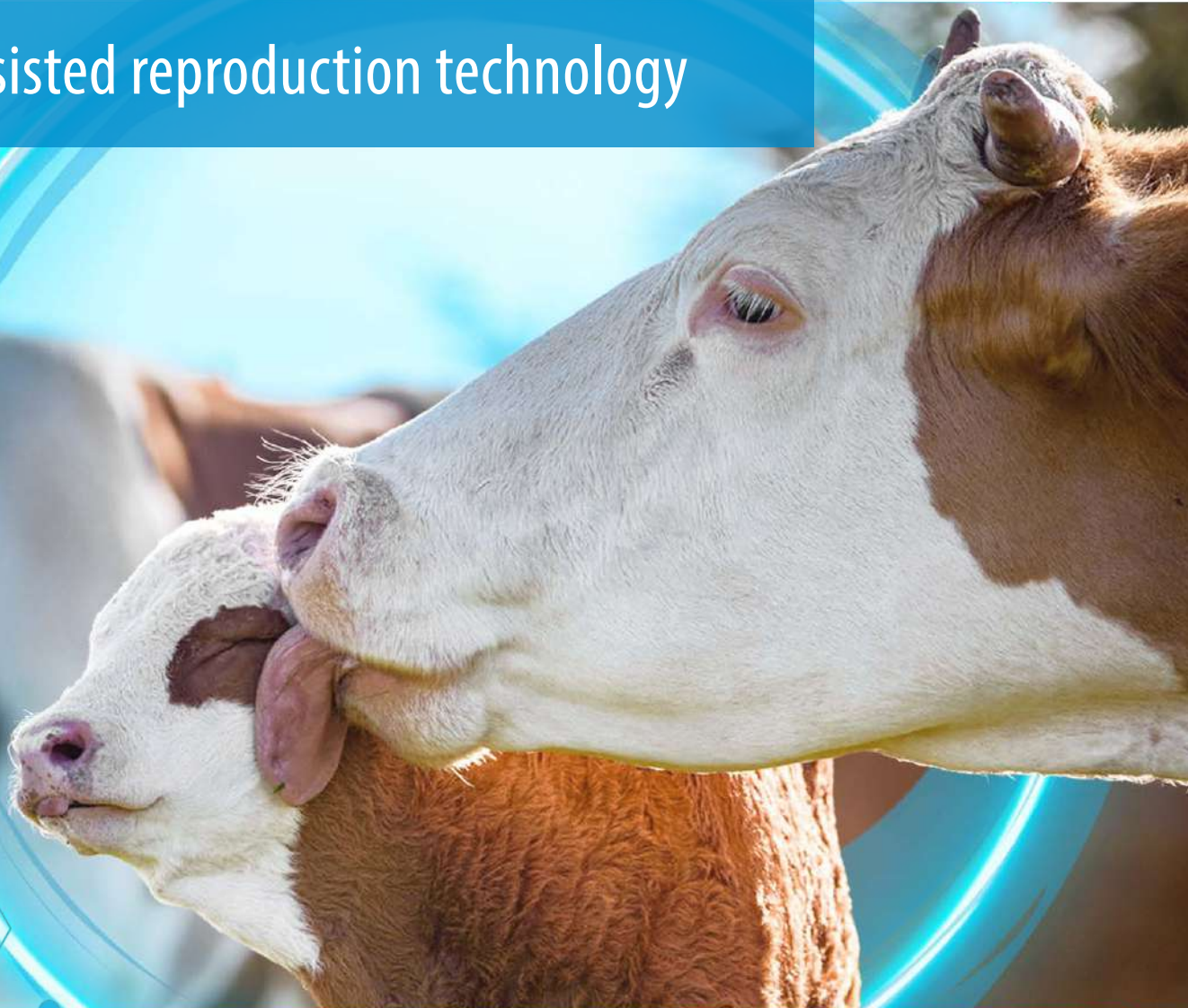

BOVINE

assisted reproduction technology



minitube
our knowledge, your success

Together for progress in assisted reproduction



At Minitube, quality, service and innovation are at the heart of everything we do – ensuring solutions that truly meet your practical needs.

As a third-generation family business, Minitube's 50-year history is built on a foundation of stability and trust. In a world focused on short-term gains, we are committed to sustainable growth and **your enduring success**.

This long-term focus is rooted in **scientific precision**. Our integrated team of veterinarians, biologists and engineers does not just create products; they develop the standards you rely on. The same experts deliver our **industry-leading service**, offering you direct access to practical application support and scientific advice.

We do not leave **quality** to chance. We take full responsibility by manufacturing our core products in our own state-of-the-art, ISO-certified facilities. Our media production is GMP-certified, and our quality control is uncompromising. This **rigorous control** is your assurance of safety, reliability, and consistent results.

Looking ahead, we remain committed to **pushing boundaries** – developing advanced products, improving workflows, and delivering exceptional technical service – **to help you succeed**, just as we have for more than five decades.

Katharina Rohrmüller
CEO





Minitube online media library:
Access all catalogs digitally,
available in multiple languages.

CONTENT

1	Laboratory management	02
2	Semen collection	04
3	Semen evaluation	08
4	Semen dilution	14
5	Straw processing and freezing	18
6	Insemination and diagnostics	33
7	Embryo transfer and OPU/IVP	39

1 LABORATORY MANAGEMENT



Electronic bull identification

The wireless combination of iMale RFID reader and iMale label printer allows **safe reading of animal ear tags** and **direct printing of barcode labels** with the animal ID in the semen collection area. The printed barcode label can be stuck on the semen collection bag/cup and can then be used to securely identify and process ejaculates in the laboratory, i.e. with Prism10 Bull.

- Available as an independent system or in addition to Prism10 Bull
- iMale: operating distance 30 cm, plus length of device 65 cm
- iMale S: operating distance 20 cm, plus length of device 20 cm
- Maximum distance for wireless data transfer from the iMale reader to the label printer is about 10 meters
- All HDX and FDX-B transponder ear tags can be read

RFID reader

iMale, with built-in quick-charger and external power supply [1] 18001/0000

iMale S, with built-in quick-charger, external power supply and carry case [2] 18001/0010

(🔗) Accessories

iMale label printer [3] 18001/0200

Label for iMale printer, 41 x 25 mm, roll with 425 labels [4] 18001/0210



Prism10 Bull lab software

With its dynamic user interface, the lab software Prism10 Bull offers a **modern process and product-based concept** for controlling bull semen production.



★ Product features

- Straightforward ejaculate processing: bull ID, ejaculate volume (scale), concentration (photometer or AndroVision®), motility (AndroVision® or manual), evaluation and collector identification and more
- Data connection to iMale, AndroVision®, SmartDispenser, MPP Quattro & MiniJet printer (CombiSystem)
- Flexible architecture - operation from any connected computer
- Product-centered approach allows for customized mapping of production processes: every ejaculate can be processed into different products, for example with different number of sperm per dose, different packaging, dose volume, extender etc.
- Flexible tool to import and update donor information from MS Excel
- Flags for marking characteristics of ejaculates, collections, and bulls (e.g. reason for discard, quality of jump)
- Adjustable limits and warnings for ejaculate data to help station manager in decision-making during production
- Live data can be accessed remotely, e.g. orders filled, progress of daily production, discarded ejaculates, etc.
- Comprehensive reporting with customizable templates and export function (PDF, MS Excel)
- Donor animal dashboard with graphs: overview of average breeding value, age, weekly available sperm output, barn usage and others
- Production dashboard with graphs: overview of semen quality, produced doses per period, per bull, per process, among others



+ Your benefits

- + Newly developed, modern, web-based software
- + Guaranteed long-term support by experienced team of developers
- + Hassle-free connection to Minitube devices
- + Secure and traceable input of all data, entry errors are minimized
- + High production reliability, operation is not dependent on a single device
- + The most important performance parameters at a glance
- + Data is accessible remotely by smartphone
- + Overview of any time frame of production
- + Statistical analysis of all production data

Prism10 lab software subscription

12535/0005



2 SEMEN COLLECTION



Electro ejaculator

Electro ejaculator set, consisting of heavy-duty carrying case, main device with remote controller and transformer/charger. Does not include probes which need to be ordered separately. Semen collection is **made easier**, especially from older bulls that may be difficult to stimulate.

- Digital display and adjustment of the voltage in 0.5 volt steps
- Fully portable with remote control
- Pulse/pause ratio adjustable manually
- Power output with 4 different curves enabling balanced setting according to the animal's size and sensitivity
- Power supply: 115 V/60 Hz or 230 V/50 Hz

Electro ejaculator [1] 11900/0000

(↻ Accessories

Electro ejaculator probe

2.5", Ø 7 cm [2] 11900/0025

2", Ø 5.6 cm [3] 11900/0022

Semen collection set for electro ejaculator [4] 11900/0250





Bovine collection dummy

Comfortable and safe semen collection for technician and bull. With hydraulic height adjustment.

- Body made of solid, zinc-coated steel
- High quality padding
- Cover made of reinforced PVC material
- Inside seat for the collector

+ Your benefits

- + Hydraulic height adjustment for bulls of different sizes
- + Forward and backward movements to stimulate the bull
- + Separated, smooth hindquarters reduce risk of injury to the bull penis
- + Comfortable and safe semen collection
- + Optimal hygiene when used with disposable sanitary covers



Bovine collection dummy	
mobile [1]	11000/1000
stationary [2]	11000/0500

+ Accessories

Disposable sanitary cover, 100/package	11000/0200
Plastic cover for dummy cow, with leather belt	11000/0100



Collection dummy for young bulls

Fixed collection dummy of small dimensions for young bulls. Height at mounting end: 105 cm (114 cm on the opposite side), length: 130 cm.

- The two feet at the mounting side are in an oblique position facilitating comfort for both the bull and the semen collector, because of leaving space for the artificial vagina
- Especially young bulls feel more comfortable with a low mounting phantom
- Also usable with older bulls
- Stable and robust steel body reinforced with durable PVC coating and thick foam padding

Collection dummy for young bulls	11000/1500
----------------------------------	------------



Bovine artificial vagina

Minitube offers bovine artificial vaginas in 4 different lengths to suit all kinds and sizes of bulls.

Artificial vagina casing with valve [1]	
20 cm	11021/0020
30 cm	11021/0033
35 cm	11021/0035
41 cm	11021/0041
Latex inner liner	
smooth, 650 mm [2]	11022/0002
rough, 650 mm	11022/0003
rough, 480 mm	11022/0005
Disposable inner liner	
roll with 200 m	11022/0010
with integrated collection vial (15 ml) and lid, sterilized [3]	11022/0011
conical form, open end	11022/0012
conical form, closed end	11022/0013
Director cone	
latex, for use with 15 ml tubes	11024/0000
silicone, for use with 15 ml tubes [4]	11024/0050
latex, for use with 50 ml tubes	11024/0051
Warming bag, neoprene, blue, 58 cm [5]	
11025/0058	
Warming bag, green [6]	
18 cm	11025/0018
35 cm	11025/0035
45 cm	11025/0045
Rubber ring	
11022/0001	



Accessories

Greasing stick [7]	11905/0050
ReprJelly, non-spermicidal lubricant, water soluble	
1 l bottle [8]	11907/1000
3 l canister, without dispenser [9]	11907/3000
Dispenser for ReprJelly canister	
11907/3100	
Thermometer for artificial vagina	
Bimetal, non-breakable, up to +100°C	11027/0000
up to +200°C, graduation 0.1°C, red filling [10]	11027/0200
Collection vial, glass, graduated, 15 ml	
11023/0015	
Collection vial, plastic, graduated, with lid, 15 ml	
11023/0018	
Semen collection vial, tulip shape, graduated, 8 ml [11]	
11023/0016	
Semen collection and AI glove, blue, nitrile, extra sensitive, sperm safe, latex-free, powder-free, 100/box [12]	
size M	11128/0055
size L	11128/0060
size XL	11128/0065



Sterivar

Low pressure sterilizer, specially designed for artificial vaginas and accessories. Made of stainless steel, double wall with insulation. With pressure safety valve, timer, water drain, protection against overheating and insert for artificial vaginas for bulls.

- Capacity: 15 artificial vaginas
- Temperature range: up to +100°C
- Pressure: max. 0.02 bar
- Inside dimensions: 750 x 400 mm
- Power supply: 115 V/60 Hz or 230 V/50 Hz
- Power consumption: 3500 W

Sterivar [1]

230 V	11050/0015
115 V	11050/0014

Accessories

Roller base for Sterivar [2]	11051/0000
------------------------------	------------

Sterivar insert for 15 cones [3]	11051/1015
----------------------------------	------------



Warming cabinet

- Suitable for artificial vaginas and accessories
- Casing and interior made of stainless steel
- Volume: 449 l, 2 insert racks
- Inside dimensions: 104 x 72 x 60 cm (W x H x D)
- Outside dimensions: 122 x 124 x 78 cm (W x H x D)
- Forced air ventilation
- Adaptive multifunctional digital PID-microprocessor controller with high-definition TFT-color display
- 2 Pt-100 sensors with two-way control
- Digital timer (adjustable from 1 minute to 99 days)
- Temperature range: +10°C above ambient up to +300°C
- Power supply: 400 V (± 10%) / 50-60 Hz
- Power consumption: approx. 5800 W (in heating function)

Warming cabinet	14135/0400
-----------------	------------



We offer many different models and sizes of warming cabinets, also pass-through models.

3 SEMEN EVALUATION



■ AndroVision®

AndroVision® is a **highly precise, automated system for computer-assisted semen analysis**. It enables fast and accurate assessment of concentration and motility, analyzing up to 1000 cells per field. When combined with a fluorescence microscope, additional software modules expand the range of analysis options.

Seamless integration with Prism10 Bull laboratory management software ensures efficient workflows and reliable data management. The intuitive user interface makes operation simple: **just three clicks from start to result**.

(★ Product features

- Real-time analysis of live images and video files
- Analysis of up to 4 fields within 20 seconds
- Highly efficient particle filter for accurate sperm identification
- Advanced image processing
- Lightmeter for illumination control
- Powerful algorithms, enhanced with artificial intelligence
- Predefined analysis profiles for many species
- Adjustable analysis parameters and customizable user profiles
- Comprehensive database with flexible reporting options
- Data export to MS Excel and other programs
- Languages: English, German, Spanish, Chinese, Portuguese, Russian, French
- Regular software updates and long-term expert support

AndroVision®, with PC and monitor **12500/0000**

AndroVision®, with PC, touch screen, barcode scanner **12500/0001**

AndroVision®, with laptop **12500/0010**

(≡ Optional software modules

Morphology and Morphometry **12500/1300**


Measures length and width of the sperm head, head shape and midpiece asymmetry of each single sperm cell (acc. to Kruger). Results can be classified into a large range of morphological abnormalities. [1]

Dose Calculation **12500/1100**

Automatic calculation of the number of doses that can be prepared from an ejaculate and of the volume of extender to add to the ejaculate.

Quality Control **12500/1200**

Analysis of samples post thaw and in holding. Link to native ejaculate analysis.

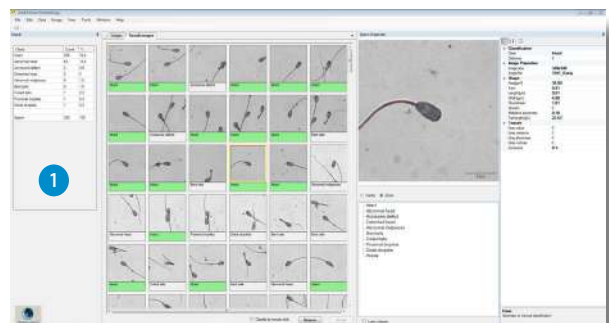



Recommended for in-depth analysis

AndroVision® with fluorescence equipment

Stands out by combining classic CASA analysis with advanced assays for sperm functionality. Acrosome integrity, viability, mitochondrial activity and DNA integrity are analyzed within seconds – making AndroVision® the perfect tool for **detailed and comprehensive semen evaluation**.

As part of laboratory quality control, these analyses not only allow continuous fertility screening of the animals, but also the early identification of premium and standard breeding animals.

(☰ Advanced software modules

Viability* 12500/1400

Automatic count of the percentage of spermatozoa with intact membranes. [1]

Acrosome Integrity* 12500/1600

Assessment of fertility by evaluating acrosome status. An intact plasma and acrosomal membrane are essential for sperm to penetrate the egg.


Mitochondrial Activity* 12500/1700

Analysis of sperm energy processes for optimal motility. By analyzing mitochondrial activity, the energy metabolism of the sperm can be assessed.

DNA Integrity* 12500/1800

Detection of DNA fragmentation as an indicator of sperm quality and fertility. The Halo technique differentiates between sperm with intact DNA and sperm with fragmented, defective DNA.

* Requires fluorescence microscope.

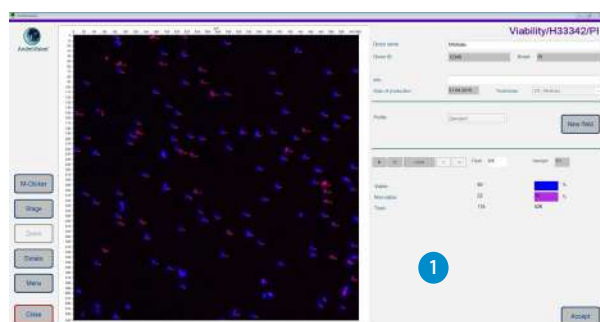

(↗ Accessories

AndroVision® high speed color camera [2] 12500/4400


Phase contrast microscope Zeiss AxioScope 5 with LED fluorescence* [3] 12008/0015

ScanStage, automated microscope stage with integrated heating system [4] 12048/0031

Controller HTi 200 [5] 12057/0200

Warming plate, 245 x 200 mm [6] 12057/0510

* Other microscopes from our portfolio may also be used. Contact us for a personalized offer.

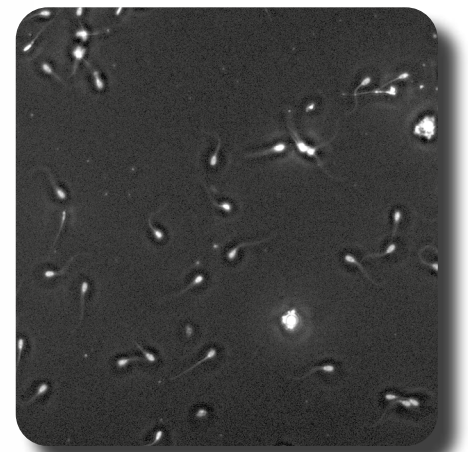

+ High image quality

AndroVision® with MiniQube

Combines the industry's leading CASA software with an **innovative digital microscope** to deliver a modern, precise, and reproducible solution for semen analysis across multiple species. AndroVision® MiniQube saves valuable time, eliminates subjectivity, and provides reliable results at the touch of a button.

+ Your benefits

- Rapid, reliable analysis: Process up to 30 ejaculates per hour
- Unmatched measurement reproducibility: Standardized results across users, labs, and production days
- Low-maintenance design: Ready-to-use with minimal setup
- All-in-one compact system: Replaces multiple devices with a streamlined design, small footprint, and reduced cabling

AndroVision® MiniQube, w/o PC and monitor 12500/5000

AndroVision® MiniQube, with PC and monitor [1] 12500/5001

MiniQube for existing AndroVision® systems 12500/5005

↔ Accessories

Controller HTi 50 [2] 12057/0100

Warming plate, 245 x 200 mm [3] 12057/0510



★ Product features MiniQube

- Fully automatic autofocus: High image quality for every user and every analysis
- Heated and motorized stage: Stable, sperm-friendly temperature and fast, accurate sample positioning
- Unique, perfectly aligned optics (patent pending): Maintenance-free, no need for manual adjustments
- Digital light adjustment: Optimal clarity and contrast without manual intervention



+ Small footprint

AndroScope

AndroScope is a compact mobile CASA system based on AndroVision®. It is designed for rapid, accurate semen analysis across multiple species at any stage of production, transport, or insemination.

★ Product features

- Equipped with a professional high-speed camera and a robust focus adjustment for precise evaluation of semen motility and concentration
- The heating unit in the analysis area reaches +37°C within a few seconds, providing optimal temperature conditions
- Analyze fresh, chilled, or frozen-thawed semen
- Follow the analysis live on the screen and generate PDF reports with key parameters clearly mapped
- Connect AndroScope to any Windows 10+ PC, laptop, or tablet via USB 3.0 for offline analysis using the installed software
- Alternatively, use the free AndroScope App (available for Android and iOS) on your smartphone for fully mobile CASA analysis
- Perform real-time analysis in the App and enjoy smooth live video streaming to your smartphone via the WiFi station
- The WiFi station creates a dedicated wireless network and can be powered by a wall outlet or portable power bank
- Intuitive interface requires minimal training while delivering reliable results
- Robust carry case provides optimal protection during transport and contains everything needed for semen analysis

AndroScope [1] incl. transport case and accessories 12500/3000

AndroScope Package: AndroScope, WiFi station, mobile warming unit, transport case and accessories 12500/3100

🔗 Accessories

Disposable counting chamber Minitube, 20 µm, 4 chambers, 25/box 12050/0220

Test tube 2 ml with cap, 250/bag 14602/0904

Disposable capillary tube, 100/bag 15404/0100

Micro fiber rod, length 124 mm, 100/bag 12510/0106

For mobile operation on smartphone: AndroScope WiFi station [2] 12500/3010

Recommended for sample preparation: Mobile warming unit, for 6 reaction vials and 1 slide [3] 12057/6000



AndroScope App

Available for free on Google Play and App Store.



Phase contrast microscope CX 43

- Phase achromat optics with universal condenser
- Heated microscope stage with temperature control unit HTi 200 and additional warming plate (180 x 180 mm)
- Integrated LED illumination
- Ergonomic design

Phase contrast microscope CX 43, with phototube (trinocular), objectives 20x, 40x **12005/0042**



Phase contrast microscope MBL

Functional microscope for semen analysis with heating system and control unit for freely adjustable temperature.

Phase contrast microscope MBL 2000

binocular, heated stage, HTi 50, objectives 10x, 40x, 100x **12010/3000**

binocular, heated stage, HTi 50, objectives 10x, 40x, 100x, with camera and monitor [1] **12010/3050**

Phase contrast microscope MBL 2100

trinocular, heated stage, HTi 50, objective 20x **12010/3103**

trinocular, heated stage, HTi 200, objective 20x, with camera and monitor **12010/3101**


Digital color camera for microscopes [2]

with integrated image display software **12110/1600**

Monitor 11.6" with HDMI input [3]

12110/1610

Wide range of microscopes and accessories available

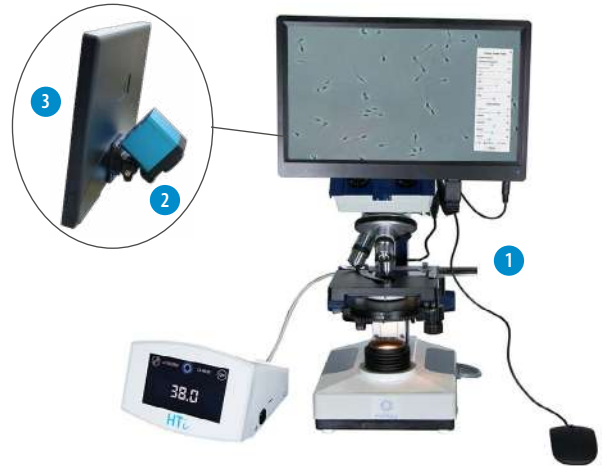

Semen stains

Spermac, stain for morphology analysis, 4 x 50 ml [1] **15405/0000**

Farely stain, for morphology analysis, 3 x 250 ml [2] **15405/0026**

Eosin G, 2% solution for supravital staining, 50 ml [3] **15405/0025**

Nigrosin, 4% solution for supravital staining, 50 ml [4] **15405/0029**



Photometer SDM 6

- Determination of sperm concentration and calculation of semen doses for boars, bulls, stallions, rams and bucks
- Automatic calculation of extender volume and number of doses
- Display of semen concentration in 10^9 sperm/ml
- Large touch screen for easy handling
- Serial interface for data transfer
- Integrated printer for printing ejaculate production data

Photometer SDM 6 12300/0006

Accessories

Control cuvette set, 4 pieces 12300/0010

Paper for printer 12300/0311

Macrocuvette, 4 ml, 100/package, disposable 12301/0003

NaCl solution, 0.9 %, 1 l 12301/4667

Dispenser, 0.5-5 ml for NaCl solution 12428/1815

Bottle for dispenser, 500 ml 12428/1905

Adjustable single channel pipette, yellow, autoclavable, 10-100 μ l 12425/4810

Pipette tip, 10-100 μ l, yellow, 1000/bag 12425/9409



Proven accuracy

SMD 6 and SDM 1 are compact, highly precise photometers for measuring sperm concentration. Cleaning is simple, maintenance is minimal.

Photometer SDM 1

- Calibrated for six different species, including porcine, bovine, equine, canine, caprine and ovine
- Raw semen samples can be used without dilution
- Semen concentration is displayed in 10^6 sperm/ml
- Electrical connection: with included mains adapter 100-240 V or batteries

Photometer SDM 1 [1] 12300/0200

Microcuvette for SDM 1, 100/box, disposable [2] 12300/1111



4 SEMEN DILUTION



Water bath

- Stainless steel, with angular lid
- Electronic temperature control
- Working temperature range: minimum 5°C above ambient up to +100°C (depending on ambient pressure)
- Independent temperature limiter to switch off the heating in case of failure
- Intuitive 3.5" color graphic display with touch screen

Water bath, 51 l, 230 V/50-60 Hz

14082/0045



Different sizes of water baths available



Electronic precision scale

- Weighing range: 2000 g
- Resolution: 0.1 g
- Weighing pan: 130 x 130 mm

Electronic precision scale up to 2 kg

14295/0442



Magnetic stirrer

For stirring of semen extender. With connection for contact thermometer.

Magnetic stirrer with heater [1]

115 V/60 Hz 14235/0731

230 V/50 Hz 14235/0732

- Heating capacity: 800 W
- Temperature range: +20 to +300°C
- Speed: up to 1400 revolutions per minute
- Maximum load: 20 l

Magnetic stirrer without heater [2]

115 V/60 Hz 14230/0001

230 V/50 Hz 14230/0000

- Speed: up to 2200 revolutions per minute
- Maximum load: 5 l

Contact thermometer, electronic 14235/7320

Support for thermometer 14235/7323

Stir bars

30 mm 14240/7130

40 mm 14240/7140



SmartDispenser 3.0 S

Intelligent combination of devices ensures accurate extender dosing (± 1 g) and facilitates dispensing of smaller volumes. It helps to achieve very precise sperm cell numbers per dose and an efficient workflow in the semen laboratory.

- Automatic dilution of ejaculates with the desired extender volume
- Assembly includes pump unit with integrated controller, scale, stand to hold the tubing, and required tubing with sinker
- High dilution accuracy: constant feedback between pump and controller
- Connected workflow: can be combined with a heated extender vat and Prism10 Bull lab management software
- Simple operation and easy navigation with touch display
- Small footprint

SmartDispenser 3.0 S, scale up to 2 kg, resolution ± 1 g, pump flow rate up to 1.2 l/min 13200/0410



AndroMed®

- Perfect biosecurity: no ingredients of animal origin
- Constant, optimal freezing results, and high fertility rates with every dilution ratio
- Increased efficiency of microscopic analysis: extremely clear cell images
- Reduced preparation time

AndroMed®* 13503/0200
200 ml, for a total of 1000 ml of extender

AndroMed® CSS, One-step** 13503/1200
200 ml, for a total of 1000 ml of extender

AndroMed® CSS, Two-step**

Fraction 1, 100 ml, for a total of 500 ml of extender	13503/1250
Fraction 2, 200 ml, for a total of 500 ml of extender	13503/1251



Egg-free media for semen freezing

BoviFree

- Constant, optimal freezing results with all equilibration protocols
- Enhanced post-thaw semen parameters
- Brilliant microscopic images due to low viscosity and homogenous particle-free consistence
- Very easy to prepare

BoviFree* 13501/0200
200 ml, for a total of 1000 ml of extender

BoviFree CSS, One-step** 13501/1200
200 ml, for a total of 1000 ml of extender

BoviFree CSS, Two-step**

Fraction 1, 100 ml, for a total of 500 ml of extender	13501/1250
Fraction 2, 200 ml, for a total of 500 ml of extender	13501/1251



Egg-free media for semen freezing

Steridyl

- Elaborate, proven buffer formula
- Constant, optimal freezing results with every dilution ratio and for most bulls
- Easy to prepare, just add water and save time
- Sterilized, requires no addition of egg yolk

Steridyl* 13500/0260
500 ml, for a total of 1250 ml of extender

Steridyl CSS, One-step** 13500/0280
500 ml, for a total of 1000 ml of extender

Steridyl CSS, Two-step**

Fraction 1, 500 ml, for a total of 1000 ml of extender	13500/0290
Fraction 2, 500 ml, for a total of 1000 ml of extender	13500/0295



Media for semen freezing with egg yolk

Triladyl®

- Internationally established for over 50 years
- Elaborate formula, based on TRIS
- Ready to use after the addition of water and egg yolk
- Reliably optimal and constant post-thaw results with all groups and types of bulls

Triladyl®* 13500/0250
250 g, for a total of 1250 g of extender

Triladyl® CSS, One-step** 13500/1200
200 g, for a total of 1000 g of extender



Media for semen freezing (One-step)

Biladyl®

- Based on the proven Triladyl® formula
- Allows first dilution without glycerol
- Used with various freezing protocols

Biladyl®, Fraction A** 13500/0004
49 g, for a total of 500 g of extender

Biladyl®, Fraction B** 13500/0006
250 g, for a total of 750 g of extender



Media for semen freezing (Two-step)

Antibiotic supplement

- Antibiotic formula in powder according to CSS protocol
- Requires the addition of water

Antibiotic supplement for One-step extender: 13504/9000
550 mg Gentamicin, 110 mg Tylosin, 330 mg Lincomycin, 660 mg Spectinomycin

Antibiotic supplement for Two-step extender: 13500/0005
300 mg Gentamicin, 60 mg Tylosin, 180 mg Lincomycin, 360 mg Spectinomycin




* With antibiotic formula.

** Corresponds with the CSS regulations.

Maximum product security

Made at Minitube's ISO and GMP-certified extender production facility. Highest quality control standards at each production stage and guaranteed use of semen friendly components and materials only.

5 STRAW PROCESSING AND FREEZING

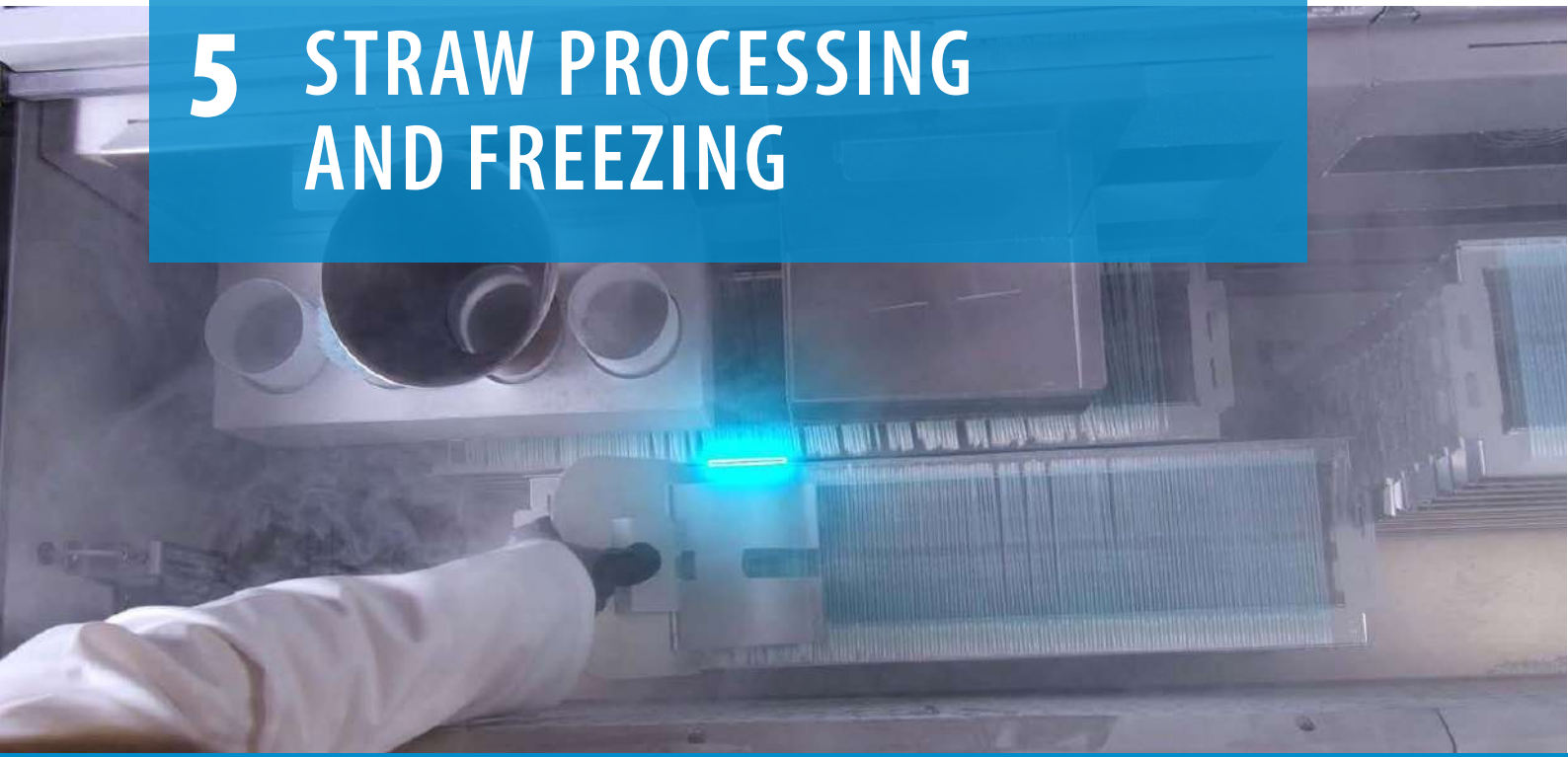

■ Semen straws

For over 30 years, Minitube straws have been used worldwide as a standard for the freezing of semen. Minitube's quality control process, production techniques, and material selection ensure for a **reliable product** that consistently exceeds industry standards.

■ Fresh semen straws and EcoStraws

Fresh semen straws are designed to process and store fresh/chilled semen at refrigerated temperatures. The wick has a special powder which does not swell and absorbs less semen.

EcoStraws feature a wick with a shorter inner wick section that **absorbs 3 to 4% fewer sperm** than the corresponding symmetrical wick of standard semen straws.



+ Your benefits

- + Yield a larger number of semen straws per ejaculate because fewer sperm are lost in the wick
- + Better use of valuable genetics without adding additional cost
- + EcoStraws are ideal for overnight equilibration



Fresh semen straws and EcoStraws

These are used in the same way as standard semen straws. Identical processes for:
filling / sealing / printing / insemination

Minitube semen straws

	MiniStraw 0.25 ml	EcoStraw 0.25 ml	Fresh semen straw 0.25 ml	Medium straw 0.5 ml	EcoStraw 0.5 ml
transparent	13407/0010	13407/3010	13407/2010	13408/0010	13408/3010
transparent - blue-white wick	13407/0011	---	---	---	---
transparent - red-white wick	---	---	13407/2212	---	---
red - transparent	13407/0044	13407/3044	13407/2044	13408/0044	13408/3044
green - transparent	13407/0054	13407/3054	13407/2054	13408/0054	13408/3054
blue - transparent	13407/0064	13407/3064	13407/2064	13408/0064	13408/3064
grey	13407/0070	---	---	---	---
grey - transparent	13407/0074	13407/3074	---	13408/0074	13408/3074
purple - transparent	13407/0084	13407/3084	---	---	---
yellow	13407/0090	13407/3090	---	---	---
yellow - transparent	13407/0094	13407/3094	13407/2094	13408/0094	13408/3094
pink	13407/0100	13407/3100	---	13408/0100	13408/3100
turquoise	13407/0130	13407/3130	---	13408/0130	13408/3130
orange	13407/0140	13407/3140	---	---	---
orange - transparent	13407/0144	13407/3144	13407/2144	13408/0144	13408/3144
salmon	13407/0160	13407/3160	---	---	---
beige	13407/0180	13407/3180	---	---	---
beige - transparent	---	---	---	---	13408/3184
pistachio green	---	13407/3200	---	---	---
pistachio green - transparent	13407/0204	13407/3204	---	13408/0204	13408/3204
white	---	---	---	13408/0210	13408/3210

Cold handling cabinet

★ Product features

- Spacious cold handling cabinet made of stainless steel for semen processing at +5°C working temperature
- Height of refrigerated area: 50 cm (34 cm below acrylic cover in the front)
- Inner surfaces all stainless steel, tiltable faceplate, tiltable plexiglass cover
- Digital temperature controller: temperature programmable from 0°C to ambient
- Even temperature distribution due to a special routing of air flow
- Automatic programmable defrosting function
- Electrical evaporation of condensate
- Interior lighting for optimal working conditions
- Two power sockets in the working area
- Two 50 mm side openings (one on each side) for cables
- Storage capacity underneath the cooling surface
- Mounted on heavy duty rolls
- Power supply: 115 V/60 Hz or 230 V/50 Hz
- Power consumption: approx. 450 W

Available in two sizes,
with or without hydraulic height adjustment

	Working area (cm - W x D)	Outer dimensions (cm - W x D x H)	230 V/50 Hz	115 V/60 Hz
Large	200 x 90	216 x 114 x 127	14335/0403	14335/0400
Large (height adjustable)	200 x 90	216 x 114 x 127	14335/0402	14335/0399
Standard	200 x 67	216 x 91 x 127	14335/0401	14335/0398
Standard (height adjustable)	200 x 67	216 x 91 x 127	14335/0404	14335/0397



CoolingCastle

★ Product features

- Outside dimensions (W x D x H)
 - Including spacers on the back wall: 1420 x 960 x 1752 mm
 - Device only, without spacers: 1420 x 890 x 1752 mm
- Inside dimensions (W x D): 1280 x 650 mm
- Electrical evaporation of condensate
- Network compatibility
- Equipped with power outlets, a network port, bright LED light and a shelf at the rear panel
- Power supply: 230 V/50 Hz or 110 V/60 Hz
- Power consumption: approx. 810 W

CoolingCastle

230 V/50 Hz

14335/0422

110 V/60 Hz

14335/0421

+ Your benefits

- + Small footprint (passes standard-sized doorways) allows use in small labs
- + Easy monitoring of the processes inside the cooling chamber due to transparent double-wing flap door
- + A curtain of cooled air minimizes air transfer into the inner chamber: even if the door is kept open, the CoolingCastle guarantees +5°C in the chamber (at room temperature)
- + Separate electric connection: even if cooling is switched off, the sockets in the CoolingCastle can be used



Intelligent design

The CoolingCastle is the ergonomic and space-saving solution for processing semen at +5°C.



MPP Quattro and MPP Uno

- Automatic filling and sealing machines for 0.25 and 0.5 ml straws
- Fast, tool-free conversion between 0.25 and 0.5 ml straws and vice versa (1-2 minutes)
- Exchangeable hopper, capacity: 2400 MiniStraws (0.25 ml) or 1200 medium straws (0.5 ml)
- Ultrasonic sealing unit, ultrasonic generator in an external casing that can be installed at a distance of up to 2.5 m (ON/OFF switch on the machine)
- Designed to provide ideal conditions for hygiene and safety
- Can be operated in cold room or cabinet or at room temperature as stand-alone or combined with MiniJet printer
- Vacuum unit with integrated reservoir and automatic readjustment function; vacuum level can be easily modified via buttons
- PLC microprocessor control
- Power supply: 110 V/60 Hz or 230 V/50-60 Hz



MPP Quattro

Filling of 1 to 4 straws at a time – freely selectable by the operator.

★ Product features

- Capacity: approx. 14000 straws/hour
- Exchangeable compact ultrasonic sealing unit
- Modern touch-screen monitor for software control and display; easy to operate, with color coded display; continuous display of number of filled straws, machine status, straw size
- Quick preparation for a new ejaculate: blocks of 4 filling and suction needles each allow for a very quick reset
- Optimal control by the operator: processing of 4 straws at a time can easily be monitored
- Semen-saving: when ejaculate is almost finished, active filling needles are automatically reduced from 4 to 1 and residual semen from inactive needles runs back into reservoir - practically no semen is wasted, and the ejaculate is used to the very last drop
- Fills the first 8 straws of each ejaculate with reduced vacuum and increased filling time to ensure optimal filling
- Memory function allows storing up to 8 sets of filling and sealing parameters: easily select the optimal parameter set for each straw-extender combination
- Machine dimensions: 650 x 390 x 500 mm (W x H x D) plus ultrasonic generator



Operation security

Very quick and easy exchange of the compact sealing unit ensures reliable, uninterrupted operation in case of service.

MPP Quattro, complete

13018/0000

MPP Uno

Seals and fills 1 straw per cycle – ideal for handling small volumes.

★ Product features



- Capacity: approx. 4000 straws/hour
- Very compact device which takes up little space
- Machine dimensions: 460 x 350 x 410 mm (W x H x D) plus ultrasonic generator

MPP Uno, complete 13017/0000



🔗 Accessories for MPP Quattro and MPP Uno

Filling block with 4 nozzles for MPP Quattro [1]	
for 0.25 ml straws	13018/0012
for 0.5 ml straws	13018/0017
Suction block with 4 nozzles for MPP Quattro [2]	
for 0.25 ml straws	13018/0022
for 0.5 ml straws	13018/0027
Filling nozzle for MPP Uno	
for 0.25 ml straws	13017/0012
for 0.5 ml straws	13017/0017
Suction nozzle for MPP Uno	
for 0.25 ml straws	13017/0022
for 0.5 ml straws	13017/0027
Straw hopper, for 0.25 and 0.5 ml straws [3]	13018/0006
Washer for 0.25 ml and 0.5 ml nozzles, 5000/bag	13021/3001
Filling and suction tubing, 380 mm, sperm friendly material (silicone), angle cut edge, 100/bag	13021/3800
Disposable semen cone, 100/bag [4]	13018/0100
Fixing unit for filling and suction blocks, stackable, stainless steel [5]	13017/0050
Straw seal tester for 10 straws per cycle (0.25 ml / 0.5 ml)	13019/0210

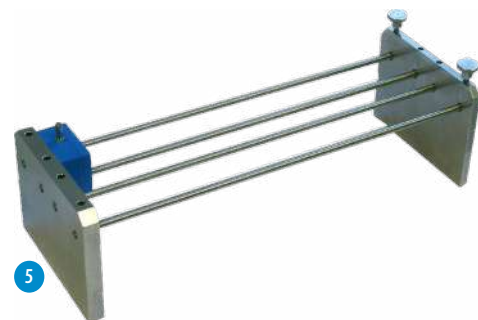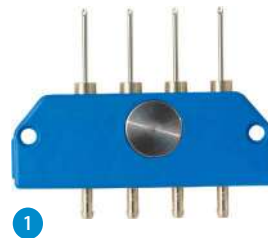



MiniJet

Professional printing system for 0.25 ml and 0.5 ml straws. Combines excellent print quality with high print speed.

+ Your benefits

- + Immediately abrasion-resistant print (character size 1.4 mm to 5 mm; bold print option; printing of barcodes and logos)
- + Minitube PrintControl software with clear and easy to use menu navigation
- + Easily adjustable print head position
- + Change of the transport belt without tools
- + No gear pump is used for generating pressure in the ink circuit, therefore no abraded particles are ageing the ink
- + Sealtronic print head technology rids clogging and frequent head cleaning: the ink is 100% protected from drying out
- + Windows-based touchscreen display for convenient operation
- + Steady once per day tuning of ink viscosity for optimal start-up conditions
- + Low operational cost: ink tank is only refilled instead of replacing it altogether
- + Data input from customer databases for easy customer specific databases
- + Interface to Minitube lab management software
- + Remote maintenance possible

Each system consists of:

- Inkjet printer Leibinger Jet3up: 490 x 450 x 290 mm (W x H x D)
- PC with Minitube PrintControl software
- Straw transport unit with optical sensor and mounted print head with exchangeable hopper: 730 x 500 x 260 mm (W x H x D)
- Control box of the transport unit: 310 x 450 x 130 mm (W x H x D)

★ Product features

- Capacity: approx. 20000 straws per hour
- Adaptable for the CombiSystem
- Power supply: 110-230 V / 50-60 Hz

MiniJet with Inkjet System Leibinger

13034/0001

↻ Accessories

Ink, 950 ml

13033/0010

Solvent, 950 ml

13033/0012

Slide for straw output

13034/0010



+ One-of-a-kind technology

Keeps the ink homogenous and ensures that ink does not dry out, even during lengthy breaks in production.

The MiniJet printing system can be combined with the filling and sealing machine MPP Quattro or MPP Uno into a CombiSystem.



CombiSystem

For highest efficiency in semen laboratories – combine your filling and sealing machine MPP Quattro or MPP Uno with the MiniJet printing system for a perfect automatic integration of semen processing: filling, sealing, and printing of straws in one system and one step.

In addition to the features and benefits offered by the individual devices, the combination of MPP Uno or MPP Quattro with MiniJet provides even more advantages.

CombiSystem, 110 V/50-60 Hz or 230 V/50-60 Hz	
composed of MPP Quattro and MiniJet capacity: 14000 straws/hour	13018/0002
composed of MPP Uno and MiniJet capacity: 4000 straws/hour	13017/0002

+ Your benefits

- + Perfect automatic integration of semen processing: filling, sealing, and printing in one system and one step
- + Fully electronic control of all components
- + Semen level control and automatic stop of straw supply at the end of the ejaculate
- + Includes software with bull data files, can be combined with Prism10 Bull laboratory software
- + Requires no extra control unit, only standard PC (included)
- + Innovative PrintControl print software with intuitive user interface and preview functionality for easy control of print results
- + Diagnostic program for connected devices



Straw processing and freezing

EasyCoder



Clean and odorless printing of straws without ink and solvent.

- Thermal transfer printer with automatic straw supply (output: up to 3600 straws per hour) for immediately abrasion-resistant print
- Unparalleled precise and clear printing on 0.5 and 0.25 ml straws; easy and quick conversion between different straw sizes
- Prints up to 3 lines per straw, including barcodes, logos, and special characters
- Control via PC by a clear and user-friendly Minitube software
- Easy handling, little maintenance required
- Dimensions (L x W x H): 660 x 370 x 310 mm (PC not included)
- Power supply: 110 V/50-60 Hz or 230 V/50-60 Hz

EasyCoder printer	13038/0000
Thermal transfer ribbon, black	13038/0010



MultiCoder



Professionalize your lab process with quick and easy machine-readable labeling of samples.

- High resolution inkjet printing
- Print head protection system (avoids drying out of ink)
- Wide variety of product carriers
- No tools needed for change of product carriers
- Pre-designed print layouts
- Loads excel files for printing
- Easy drag & drop editing of print layouts

+ Your benefits

- + Very versatile: print system can label most kinds of materials such as plastics, glass, or aluminum
- + Manually moveable slide and a unique, easy-to-use carrier system allow for a speedy work process
- + Easy printing of information on the smallest amount of space, including text, barcode, QR-Code, and logo

MultiCoder printer	13145/0000
--------------------	------------



Unique, flexible, versatile

Quick and easy print on a whole range of items: semen straws, goblets, cassettes, culture dishes, macrotubes, syringes, cryovials and many more.

ScanDMC

Smartphone holder that turns a standard smartphone into a reliable, low-cost data matrix code scanner for 0.25 ml or 0.5 ml straws.

- Holds straws at the ideal scanning distance
- Eliminates reflections to improve accuracy
- Supports all Minitube semen straw colors
- Shields from ambient light and uses LED backlight (6 colors) for optimal illumination
- Includes a short double USB-C cable for power supply
- Compatible with all smartphones with a USB-C port
- Adjustable to fit any phone size and camera position



ScanDMC smartphone holder **13034/0020**

SFS 133

- Straws are filled and sealed 6 by 6
- Sealing is done securely and hygienically with metal or glass sealing balls (available in different colors)
- An air bubble of 1 cm is automatically produced in each straw
- Capacity: approx. 2000 straws/hour

Semiautomatic filling and sealing machine SFS 133, with vacuum bottle, filling, and suction head [1]

for 0.5 ml straws	13133/0133
for 0.25 ml straws	13133/0135

Low density vacuum pump, 30-60 Hg, with manometer and valve [2]

230 V/50 Hz	13133/0267
115 V/50 Hz	13133/0268

Cartridge loading device, with loading block and 10 cartridges for 36 straws [3]

for 0.5 ml	13133/0005
for 0.25 ml	13133/0025

Accessories

Filling head with 6 nozzles [4]	13133/1000
Suction head with 6 nozzles [5]	13133/2000
Washer for all filling and suction nozzles, 5000/bag	13021/3001
Silicone tubing for SFS, transparent, 3 x 1 mm, length 10 m	13133/3550
Metal sealing ball	
for 0.25 ml straws	13400/9970
for 0.5 ml straws	13400/9900
Glass sealing ball for 0.25 ml straws, available in different colors	13400/997x



GobletPacker

Provides an accurate and reliable solution for filling frozen 0.25 ml straws automatically into goblets in liquid nitrogen/nitrogen vapor.

- Packs up to 100 straws/minute into a wide range of goblets (diameter: 7.1, 9.2, 10, 12, 13, 35, 53 and 65 mm)
- GobletPacker lift allows efficient and easy adjustment to changing LN₂ level within the vat
- Liquid nitrogen vat for the GobletPacker provided with a drain to recover LN₂ after the packaging task
- Acoustic signals alert user of required action
- Determine an unknown number of straws (inventory function) or count a user-defined number
- Easy to control via user-friendly touch display

+ Your benefits

- + Time-consuming packaging task is automated, no more manual labor
- + Less risk by reducing handling of liquid nitrogen, more safety for lab personnel
- + Protects semen quality by processing in liquid nitrogen/nitrogen vapor and ensuring ideal storage temperature without fluctuations
- + Accurate and reliable operation due to intelligent, sensor-based design and specialized materials suitable for use in LN₂

GobletPacker, 100-240 V [1] **16825/0000**

+ Accessories

GobletPacker kit*, includes one size goblet cartridge, goblet funnel unit, 2 goblet wheels **16825/002x**

Goblet cartridge* for one goblet size **16825/004x**

Liquid nitrogen vat for GobletPacker**, volume 150 l, height not adjustable **16826/0010**

Liquid nitrogen vat for GobletPacker***, volume 150 l, electrically adjustable height [2]

110 V **16826/0011**

230 V **16826/0023**

Lift for GobletPacker **16825/0005**

* Available for 7.1, 9.2, 10, 12, 13.5, 35, 53, or 65 mm goblets.

** Inner dimensions: 58 x 65 x 40 cm (W x D x H), total size: 78 x 100 x 106 cm (W x D x H).

*** Inner dimensions: 58 x 65 x 40 cm (W x D x H), total size: 78 x 100 x 96-126 cm (W x D x H).



Unprecedented efficiency

Processing speed of up to 100 straws per minute, combined with uninterrupted operation and parallel reloading of goblets.



StrawPal

Innovative, camera-based system for counting straws in liquid nitrogen. The StrawPal revolutionizes frozen semen straw inspection and inventory by integrating advanced camera technology and software. It accurately counts up to 800 straws within seconds, greatly improving efficiency. The straws remain securely stored in liquid nitrogen during the process, ensuring sperm quality is preserved.

The system includes: StrawPal, Mini PC, touchscreen monitor, keyboard, mouse, dewar vessel, goblet holder.


- Dimensions (StrawPal without accessories):
490 x 750 x 505 mm (W x D x H)
- Monitor can be placed on top of the StrawPal, we recommend placing the keyboard and mouse on the lab bench
- Weight: 35 kg
- Dewar volume: 3 l
- Depending on handling, liquid nitrogen in the dewar lasts for several hours
- Detection rate of about 99 % of straws, working with straws of all colors and both 0.25 ml and 0.5 ml straws

+ Your benefits

- + Ideal for inventory, monitoring of incoming and outgoing frozen semen straws
- + Preservation of sperm quality is guaranteed at all times
- + Straws of all colors, in all sorts of goblets and both 0.25 ml and 0.5 ml straws can be counted
- + Main data can be imported via barcode
- + Option to import lists of bulls and AI stations via CSV file
- + Counts are saved in a database
- + Option to create reports and packing lists
- + Easy and intuitive operation
- + Minimal cleaning required

StrawPal, 100-240 V

16824/0000

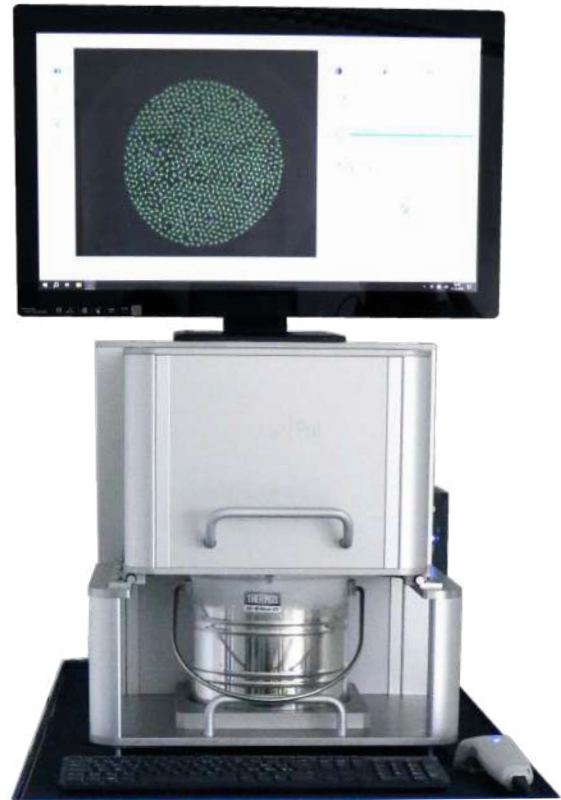

+ Fast, safe and accurate

Counts up to 800 straws within seconds and detects about 99 % of straws (all colors, 0.25 and 0.5 ml).

TurboFreezer L

- Highly insulated body with stainless steel cover
- Dual operation lid: inserted sliding covers for low impact in terms of cold loss when handling straws and racks after freezing; complete cover opens supported by gas pressure springs
- Subsequent freezing cycle can start immediately after the previous one
- Temperature curves and data of each freezing cycle can be stored, displayed, and transferred into MS Excel for further analysis
- Quadruple LN₂ injection ports, required LN₂ pressure: 2.5 bar
- Conditioning plenum separated from the freezing chamber
- 2 fan blocks with 4 fans each for homogenous LN₂ dispersion and precision gas flow throughout the chamber
- Programmable controller for custom curves
- 2 temperature sensors (1 chamber sensor, 1 sample sensor)
- Outside dimensions: 222 x 132 x 86 cm (W x H x D)
- Chamber size: 145 x 41 x 32 cm (W x H x D)
- Capacity: 30 racks
 - 175 x 0.25 ml straws (total: 5250 straws)
 - 100 x 0.5 ml straws (total: 3000 straws)

TurboFreezer L, 220 V/50 Hz (220 V/60 Hz optional) **16811/0000**



TurboFreezer M

- Provides a directional, continuous nitrogen gas flow throughout the whole chamber and therefore uniquely uniform freezing patterns
- Straw capacity: 15 racks
 - 175 x 0.25 ml straws (total: 2625 straws)
 - 100 x 0.5 ml straws (total: 1500 straws)
- Outside dimensions (without monitor and controller): 102 x 103 x 86 cm (W x H x D)
- Chamber size: 78 x 41 x 32 cm (W x H x D)

TurboFreezer M, 220 V/50 Hz (220 V/60 Hz optional) **16810/0000**



Innovative freezing technology

The TurboFreezer L is the only freezer with directed horizontal nitrogen gas flow for homogenous and superior freezing results. All straws pass through identical freezing curves!

+ Your benefits for TurboFreezer L and M

- + Precise control through user defined freezing programs
- + Ergonomic and easy handling of racks and straws in spacious freezing chamber
- + Outstanding economy due to high throughput
- + Uniform post-thaw quality of ejaculates due to homogenous semen freezing curves, independent of straw position
- + Option to start with temperature -50°C from second freezing cycle on, saving time and LN₂
- + Both freezers come with 2 holding devices for 4 goblets, straw pliers and holding box for straw pliers and funnel for filling straws into goblets (65 mm)

+ Accessories for TurboFreezer L and M

Pressure container, 189 l	16150/0185
Heating module to shorten heating time between cycles	
left side cable outlet	16811/1100
right side cable outlet	16811/1101



MT Freezer 2.0

Automatic freezer for bull semen made of stainless steel.

- Capacity: 12 racks
 - 175 x 0.25 ml straws (total: 2100 straws)
 - 100 x 0.5 ml straws (total: 1200 straws)
- Outside dimensions: 112 x 88 x 105 cm (W x H x D)
- Chamber size: 76 x 64 x 30 cm (W x H x D)
- Electronic temperature control, digital temperature display, programmable start and end temperature
- Forced gas circulation by fans
- Superior insulation
- Break- and scratch-resistant lid
- Temperature range: +5°C to -140°C
- Freezing time: 8 min (10 min for 0.5 ml straws)

MT Freezer 2.0, 230 V/50 Hz

for 0.5 bar liquid nitrogen containers **16813/0000**

for 1.5 bar liquid nitrogen containers **16813/0015**

Liquid nitrogen container

pressurized to 0.5 bar, capacity 60 l **16820/1900**

pressurized to 1.5 bar, capacity 120 l **16150/0120**

Accessories for all freezers

Liquid nitrogen connecting hose, length 2.0 m **16812/0200**

Freezing rack Turbo [1]

for 175 x 0.25 ml straws **15040/0175**

for 100 x 0.5 ml straws **15040/0100**

Loading and counting block for freezing rack Turbo [2]

for 175 x 0.25 ml straws **15041/0175**

for 100 x 0.5 ml straws **15041/0100**

Straw pliers for removal of frozen straws [3]

16811/0100

Stainless steel funnel for 65 mm goblets

15004/0402

Economic operation

Low LN₂ consumption per freezing cycle.
Enables operation of several cycles in sequence.



Goblets and accessories

Overview of liquid nitrogen containers



Goblets [1]

Ø 7.1 mm	white	16907/0071
Ø 10 mm	white*	16910/0010
Ø 13 mm	white*	16913/0133
Ø 13.5 mm	green	16914/0135
Ø 35 mm	white	16935/0135
Ø 53 mm	white	16953/0133
Ø 65 mm	white	16965/0133
hexagonal	white	16970/0136

* Other colors available.

Goblet canes [2]

for 2 x 7.1 mm goblets	16965/6171
for 2 x 10 mm goblets	16965/6000
for 2 x 13 mm goblets	16965/6013

Marking label for goblet canes, different colors 13422/00xx

Divider for goblets, 100/bag, 12 x 6,5 cm [3] 16967/0065

Identification rod, 140 mm length

for 0.25 ml straw	16903/0210
for 0.5 ml straw	16903/1210

Stainless steel funnel for 65 mm goblets [4] 15004/0402




Cassettes for straw storage

	Triangular cassette [1]*		Flat cassette [2]**	
	long	short	long	short
green	16981/01351	16981/01201	16980/1201	16980/0601
blue	16981/01361	16981/01211	16980/1202	16980/0602
yellow	16981/01371	16981/01221	16980/1203	16980/0603
red	16981/01381	16981/01231	16980/1204	16980/0604
white	16981/01391	16981/01241	16980/1205	16980/0605
beige	16981/01401	16981/01251	16980/1206	16980/0606
orange	-	-	16980/1207	16980/0607

* For 10/20 straws 0.5 ml or 28/56 straws 0.25 ml.

** For 3/6 straws 0.5 ml or 6/12 straws 0.25 ml.

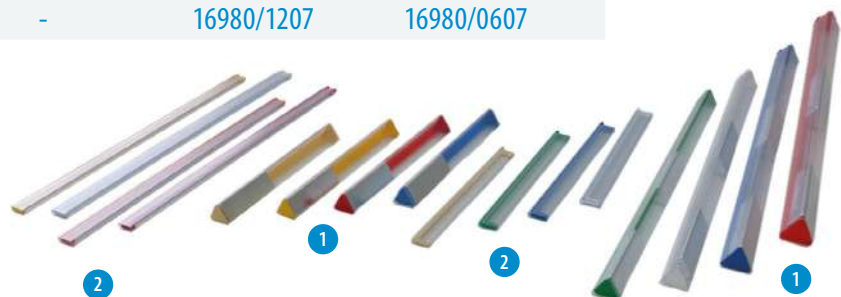

6 INSEMINATION AND DIAGNOSTICS



Insemination kit

Contains: heavy duty case, MiniThaw thawing unit, QuickLock 2000 AI gun, 100 Universal sheaths, 80 hygienic sheaths for AI gun, ReproJelly lubricant, Varikon speculum, 1 box AI gloves, MiniCutter, tweezers and clamp.

Insemination kit 17050/0070

Components also available individually:

Insemination case	17050/0000
MiniThaw thawing unit (page 35)	17048/0001
QuickLock 2000 (page 34)	17026/0000
Universal sheath non-split (page 35)	17007/0000
Hygienic sheath, roll with 80 units (page 46)	17028/0100
ReproJelly, non-spermicidal lubricant, water soluble, 100 ml tube, resealable	11907/0100
Varikon speculum	17064/0380
AI/exploratory gloves, sensitive, blue, 5 finger, shoulder length, 100/box	17080/0010
MiniCutter (page 35)	17062/0010
Straw tweezers, angular shape	17060/0000
Clamp	17063/0000



QuickLock AI gun

The AI gun for AI success, security, and accuracy.

(+ Your benefits

- + All-purpose gun for all types of straws and sheaths, split or non-split
- + Unique design for safe and centered straw alignment; no waste of semen
- + Stainless steel and sterilizable parts only
- + One hand only sheath fixation and release
- + No loose parts; safety catch prevents the plunger from sliding out
- + Identification with different colored stylets
- + Separate straw release for control of information after AI

QuickLock Classic, easy sheath fixation and release at the push of a button [1]

white	17025/0000
red	17025/0003
blue	17025/0005
yellow	17025/0008
green	17025/0009

QuickLock 2000, ergonomic design, only 1 cm of the sheath length is required for fixation [2]

white	17026/0000
red	17026/0003
blue	17026/0005
yellow	17026/0008
green	17026/0009

QuickLock Ergo, ideal for older cows due to greater usable length, smallest ergonomic handle available, easiest gun to clean [3]

white	17024/0000
red	17024/0003
blue	17024/0005
yellow	17024/0008
green	17024/0009

Protecting tube, for QuickLock AI gun and sheath [4]



Universal sheaths

- Absolutely tight fit with all types of semen straws
- Gentle insemination due to the soft shape of the tip
- Transparent adapters for better visual control of straw position
- Bags with adhesive tape for easy resealing and reopening

Universal sheath split (transparent-green) [1]

50/bag	17007/0001
with soft hygienic sleeve, 50/bag	17007/0012
individually wrapped, 50/bag	17007/0014

Universal sheath non-split (transparent-blue) [2]

50/bag	17007/0000
with soft hygienic sleeve, 50/bag	17007/0011
individually wrapped, 50/bag	17007/0013



MiniThaw thawing unit

- Variable temperature setting (ambient to +50°C)
- Digital display of current temperature
- Straw lift has three compartments and allows thawing of up to 10 straws at a time (0.5 ml or 0.25 ml)
- Connects to on-board 12–24 V socket and to mains 110–230 V socket via USB-C cable or adapter
- Robust and easy to handle

MiniThaw with USB-C cable [1]	17048/0001
-------------------------------	------------

(↻ Accessories

USB car adapter to connect to on-board 12–24 V socket	17048/0420
---	------------

USB mains converter to connect to mains 110–230 V socket	17048/0430
--	------------

Base for MiniThaw, metal [2]	17048/0005
------------------------------	------------

Digital timer, magnetic, battery included [3]	17048/0006
---	------------

Power bank, for mobile use, provides power for up to 15 hours [4]	17048/0410
---	------------



Insemination and diagnostics

MiniCutter

- Precise cutting of 0.25 and 0.5 ml straws
- Facilitates straight cut at exactly the right length, which helps to avoid semen loss
- Ergonomic shape and easy to keep clean

MiniCutter for straws	17062/0010
-----------------------	------------



Deep IU insemination

- Long, flexible sheath and stylet for insemination deep inside the uterine horn
- Metal tip of the sheath for easy palpation and exact fit with the straw
- Extra introduction stylet for penetrating the cervix

DIUI set for bovine AI and ET (using 0.25 ml straws), introduction stylet, flexible AI/ET stylet, finger support and two fixation rings	17015/0000
DIUI sheath, split, for 0.25 ml straws, 10/bag	17015/0650
DIUI sheath, split, for 0.25 ml straws, 1/bag, sterilized	17015/0660



QuickLock Heater 4.0



+ Your benefits

- + AI and ET devices are transported at a constant temperature
- + Keeps semen/embryo at optimal temperature from thawing to insemination/transfer
- + Target temperature inside the bag is freely adjustable
- + Power bank as source of electricity for more flexibility and reliability
- + Very robust and easy to clean bag material
- + Strap allows different ways of carrying the bag

QuickLock Heater 4.0	17028/0400
QuickLock Heater 4.0 with power bank	17028/0401
Accessories	
Power bank for QuickLock Heater 4.0	17028/0410
USB car adapter to connect to on-board 12-24 V socket	17048/0420
USB mains converter to connect to mains 110-230 V socket	17048/0430
Inner bag for QuickLock Heater, 100/package	17028/0201



AI/exploratory gloves

Recyclable polyethylene gloves that are both flexible and strong, giving you excellent feel combined with good protection. The reduced diameter of the glove will help to keep it on your arm without compromising its robustness.

AI/exploratory gloves, 5 finger, shoulder length	
sensitive, blue, 100/box	17080/0010
sensitive, blue, single packed, sterilized, 50/box	17080/0250
super-sensitive, transparent, 100/box	17080/0050
super-sensitive, transparent, single packed, sterilized, 50/box	17080/0052



Ultrasound scanner Minitube 3.0 Plus

Portable ultrasound scanner for multi-species obstetrical measurements with accurate automatic calculation of gestation days.

★ Product features

- Includes 11" tablet, power supply, carrying strap, ReproJelly lubricant, and charger, all housed in a heavy-duty case
- Compatible with any Android device with USB-C port
- Supports multiple scanning modes, including B-mode, dual B-mode, M-mode, PW, and CF color Doppler with automatic parameter calculations
- Built-in battery provides extended operation (up to 6 hours) with fast charging (2-3 hours)
- Large, glare-free display ensures clear visualization of anatomical structures in all lighting conditions
- Compact, lightweight, and IPX7 waterproof

Ultrasound scanner Minitube 3.0 Plus [1]	23500/2005
Linear rectal probe, 6.5 MHz, for transrectal pregnancy detection in cows [2]	23500/2106



Insemination and diagnostics

Uterine cytology brush

Disposable set for collecting endometrial cells for cytology.

Single packed and single use, sterilized, with hygienic cover and slide transport box, 10/bag.

Uterine cytology brush	17075/0010
------------------------	------------





Henryetta, bovine AI training model

This revolutionary training model cow offers unique training opportunities and provides more flexibility around the timing and location of AI courses.

Trainers can improve the trainees' learning success by viewing and guiding them through the large windows. The reproductive organs are life-like and removable and can be used for initial demonstrations of anatomy and AI techniques. A difficult or easy cervix can be used among many more options for training variations, and training units can be repeated as often as necessary.

With the pregnancy and palpation model AI PLUS students can train the most important steps of manual pregnancy check and ovarian diagnostics.

- Naturally sized and shaped cow model for AI training
- Body made of fiberglass and acryl glass, reproductive tract made of silicone
- Footprint: 1320 x 680 mm, height: 1400 mm, weight: 53 kg

Henryetta, Holstein design	
230 V	22400/1040
115 V	22400/1041
Henryetta, Simmental design	
230 V	22400/1050
115 V	22400/1051

The Simmental design is a special edition produced exclusively on request. Orders may therefore be subject to longer delivery times.



AI training at a new level

Benefit from the proven advantages for AI training and enhancement of animal welfare!

Accessories

Uterus	22400/1167
Vaginal tube	22400/1168
Cervix	
small opening for advanced training	22400/1164
wide opening for training start	22400/1163
Rectum bag	22400/1061
Cervix spoon, for easy insertion of cervix	22400/1161

Accessories AI PLUS

Pregnancy tract	
pregnancy signs approx. day 42	22400/1171
pregnancy signs approx. day 60	22400/1172
Intestine cushion	22400/1179
Ovaries	
small follicles	22400/1181
large follicle	22400/1182
corpus luteum	22400/1185
cystic	22400/1189

7 EMBRYO TRANSFER AND OPU/IVP

Temperature control

Control unit HTi 50 for heated stage or bead bath, adjustable temperature, digital display.

Control unit HTi 50 [1] 12057/0100

Heated stage, placed on top of the light base of a stereo microscope, 180 x 180 x 10 mm, requires a control unit.

Heated stage [2] 12057/0600

Shelf system to organize the laboratory working bench and optimize space usage.

Shelf system for HTi control units and small warming plates (1 pcs.) [3] 12057/0080

Bead bath, temperature-controlled incubator for sample tubes or straws of different sizes (up to 15 mm diameter) with steel balls inside for secure fixation. Usable height: 135 mm. Temperature range between ambient and +55°C. Requires a control unit.

Bead bath [4] 12057/5000

Laboratory heating plate, heated area: 470 x 260 mm, preset temperature (+37°C), with integrated control unit and external power supply.

Laboratory heating plate [5] 12055/0010



BoviFlush

Ready-to-use medium for bovine embryo recovery. Complete formulation containing buffers, BSA or PVA and antibiotics.

- Available in plastic bottles (fully recyclable) and infusion bags
- Infusion bags with 3-way port configuration and sealed ports for hygienic fluid removal [1]
- Transparent bottles with screw cap for instant visual inspection and secure, simple handling [2]
- GMP-certified production in a dedicated clean room with stringent production and quality control standards

BoviFlush with BSA	
1 l, bag [1]	19982/6001
2 l, bag [1]	19982/6002
1 l, bottle [2]	19982/6004
BoviFlush with PVA	
1 l, bag [1]	19982/6011
2 l, bag [1]	19982/6012
1 l, bottle [2]	19982/6010



Recovery medium

BoviHold

Ready-to-use holding medium for bovine embryos. The solution contains glucose, pyruvate, BSA, amino acids, growth factors and antibiotics. Applicable for holding embryos at room temperature in ambient conditions, transporting embryos, washing and rinsing embryos, loading embryos for transfer to recipient animal.

BoviHold	
3 bottles of 10 ml each	19982/6047
20 ml	19982/6046



Holding medium

BoviFreeze

Medium for conventional slow rate freezing of embryos. The cryopreservation solution contains a buffered solution with D-glucose, Sodium-pyruvate, BSA and antibiotics. With ethylene glycol for conventional freezing and direct transfer after thawing.

BoviFreeze, 3 bottles of 10 ml each	19982/6052
-------------------------------------	------------



Cryopreservation medium

BoviPlus Trypsin wash

Pre-freeze enzyme washing medium, satisfying IETS protocol requirements to qualify bovine embryos for export.

BoviPlus 0.25% Trypsin wash, 10 ml	19982/1300
------------------------------------	------------



Washing medium

Filters for embryo collection

- Secure fixation of the lid during collection and easy to open after collection (no screw cap)
- Integrated search dish engraved with a grid makes it easy to search and find embryos (ideal mesh size for all bovine embryo sizes)
- Also suitable for filtering oocytes after OPU
- Filter mesh cannot dry out as it remains in contact with fluid
- Excess fluid is controlled and removed through outlet tubing with attached thumb clamp (entry hole is big enough to ensure the necessary flow speed of the flushing medium)
- Connection to all types of Y-tubing is possible
- Sterilized and absolutely tight

EmSafe

Maintains a constant 60 ml of flushing fluid in the dish. Air vent in the lid and air outlet are provided with a filter which is holding embryos securely in the system.

EmSafe filter 19010/6000

MiniFlush®

Vertical recovery system with large filter surface area. Maintains a constant 20 ml volume during the entire flushing procedure. Due to its shallow design, search dish is perfectly suited for use with all microscopes.

MiniFlush® filter 19222/2000

Flushing tubing

Equipped with white and blue clamps to better distinguish outlet/inlet during the flushing process. Tubing made of very smooth, flexible material. End for filter connection requires no extra adapter. Disposable, sterilized, 1/bag.

Y-tubing for embryo collection with the EmSafe filter

Luer end 19011/0100

Foley end 19011/0200

Y-tubing for embryo collection with the MiniFlush® filter

Luer end 19011/0000

Foley end 19011/0001

Easy and safe to use


All-in-one dual-purpose filters and search dishes that require no attention during flushing procedure. Embryos are recovered directly from the filter dish.



Flushing catheters

- Flushing catheters are reusable and offer a very good price/performance ratio
- Due to their high transparency and good visibility, silicone catheters allow monitoring of the flow during flushing and easy control of cleaning
- Unless otherwise specified, all catheters have 4 ports/openings

Silicone catheter 2-way, Luer Lock, 1/bag [1]	
CH 12	19005/0100
CH 16	19005/0102
CH 18	19005/0103
CH 20	19005/0104
Silicone catheter 2-way, Foley end, 1/bag [2]	
CH 12	19005/0220
CH 16	19005/0221
CH 18	19005/0222
CH 20	19005/0223
Silicone catheter 3-way, Foley end, 1/bag [3]	
CH 16	19005/0230
CH 18	19005/0231
CH 20	19005/0232
Stainless steel tip catheter, CH 12, 6 ports, made from silicone, Luer Lock, 1/bag [4]	19006/0012
Mandrin for flushing catheter	
CH 12/16 (3-way)	19007/0011
CH 14	19007/0012
CH 15/16	19007/0016
CH 18/20	19007/0018
Cervix dilator, stainless steel, fully autoclavable, facilitates the introduction of the embryo collection catheter [5]	19001/1000

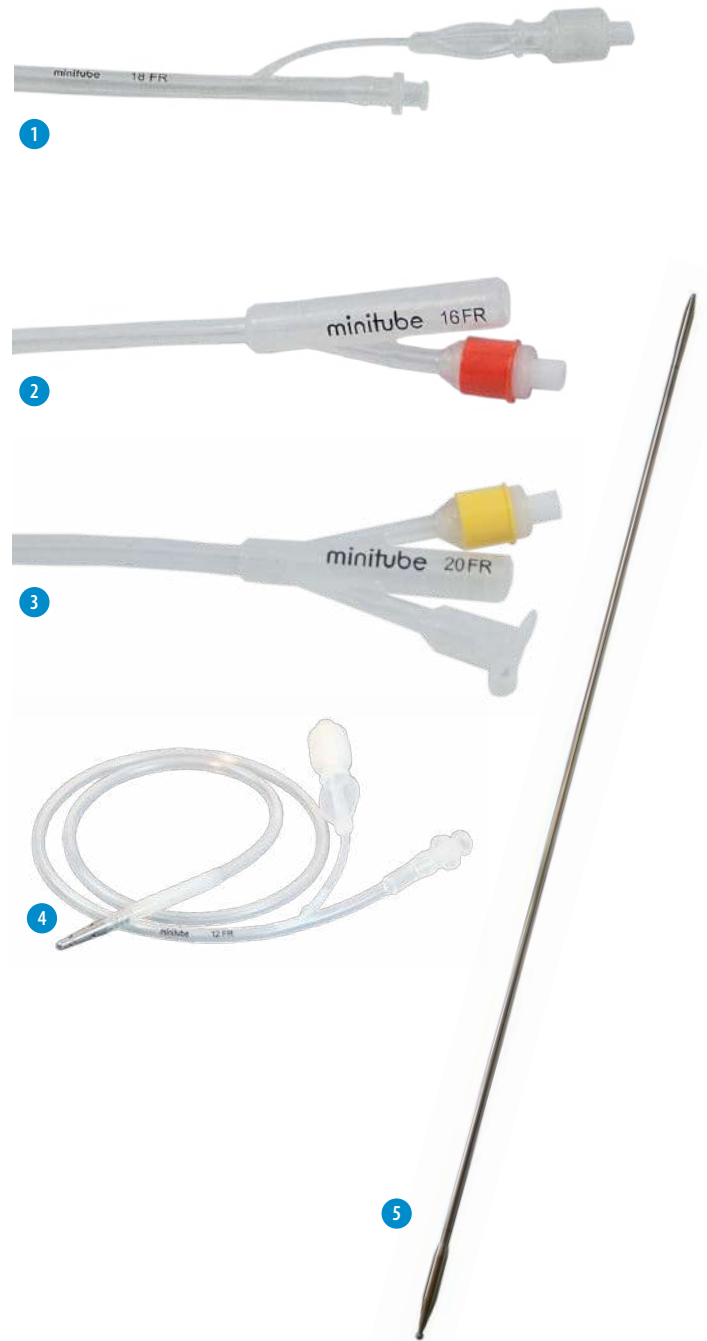

Vitrification unit

Set for vitrifying embryos or oocytes in OPS (=open pulled straws) or conventional straws.

- Styrofoam box with stainless steel vitrification bath
- Material for vitrification
- Manual with detailed description of media preparation

Vitrification unit	19500/2000
---------------------------	------------



Freeze Control® embryo freezer

Freeze Control® embryo freezing systems are designed for practitioners. They use liquid nitrogen as cooling medium, and heat control for regulating temperature.

- Each system consists of a cryobath, controller and a carry case
- Control range: +20°/-43°
- Capacity: 23 x 0.5 ml or 46 x 0.25 ml straws

Freeze Control® embryo freezer	
CL 2200, 8 preinstalled programs, 1.5 l cryobath, external power supply [1]	19406/2200
CL 5500, 16 preinstalled programs, 1.5 l cryobath, inbuilt battery power pack [2]	19406/5500
CL 8800, 16 preinstalled programs, 3.8 l cryobath, external power supply [3]	19406/8800
External power pack for CL 2200/CL 8800	19407/0000



Stereo-zoom microscope

- Wide-field HWF10x/22 mm eyepieces (no eyecups) with 45° inclined tubes
- 1:6.7 plan achromatic zoom objective, 6.7x-45x magnification, 33–4.9 mm field of view
- Anti-fungus treated, anti-reflective coated optics
- Comes with opaque and white/black stage plates and two object clamps
- 3 W LED incident and transmitted illumination with rotating mirror, adjustable separately, internal power supply

Stereo-zoom microscope, trinocular	12014/2000
------------------------------------	------------



5-well dish with lid

Embryo culture dish, rounded well edges, mouse embryo batch tested. Produced from material free of Bisphenol A and designed to facilitate pH equilibration of culture media.

5-well dish with lid, sterilized, rounded well edges, MEA tested	19021/0001
--	------------



Steripette

Steripette for embryo handling, 55 mm, individually packed, sterilized, for use with 1 ml syringe.

Steripette, 50/package	19025/0050
------------------------	------------



■ Micropipettor

Micropipettor for handling of embryos, easy and precise pipetting by handling wheel, for all sizes of capillary tubes up to 1 ml.

Micropipettor	19022/0002
---------------	------------



■ Petri dish

Petri dish, plastic, for embryo handling	
diameter 35 mm	19020/0035
diameter 94 mm	19020/0094
diameter 145 mm	19020/0145
with grid, diameter 65 mm	19020/2065



■ MiniStraw, 0.25 ml

MiniStraw, 133 mm, sterilized, 10/package	
transparent [1]	19042/0010
yellow (direct transfer) [2]	19042/1010
transparent with glass sealing ball	19041/0010
MiniStraw, 133 mm, sterilized, transparent, 10/package, with white ID rod seal (other colors available) [3]	
MiniStraw, 90 mm, sterilized, transparent, with white ID rod seal (other colors available), 10/package	19040/9010
ET identification rod, for sealing of 0.25 ml straws, 100/package, white (other colors available) [4]	
ET identification rod, for sealing of 0.25 ml straws, 100/package, white (other colors available) [4]	19046/0010
Pen for marking, very fine, alcohol permanent, black	19045/0100



Wide range of MiniStraws in different colors available



■ CryoSealer

Manual sealer using ultrasound technology. Can be used for 0.25 and 0.5 ml straws. Ultrasound welding facilitates a perfect and uniform seal of each straw. Power supply: 100-240 V AC / 50-60 Hz.

CryoSealer	13135/0000
------------	------------



Manual sealer

Manual sealer for straws, securely seals all kinds of straws by applying heat at the end of the straws (also in liquid nitrogen vapor). Consists of a plier welding head adapted for straws and a control unit for varying heat level.

Manual sealer 13135/2500



Portable incubator

- Safe and temperature-stable transport and holding of oocytes, embryos and reproductive tissue
- Constant temperature distribution by using metal balls as a transfer medium in the device
- Highly precise temperature control (+/- 0.5°C)
- Easy and secure fixation of vials and straws
- Absolutely dry and hygienic environment
- Various accessories for flexible power supply options

Portable incubator with adjustable temperature 19180/0003
(from ambient to +50°C) [1]

Accessories

Power bank, for mobile use [2] 17048/0410

USB car adapter, to connect to on-board 12-24 V socket [3] 17048/0420

USB mains converter, to connect to mains 110-230 V socket [4] 17048/0430

Base for portable incubator, metal [5] 17048/0005



Embryo transport kit

- Carrying case with portable incubator, rechargeable battery and battery charger
- For safe transport over long distances with an optimum environment
- Long lasting rechargeable battery up to 20 h
- Constant temperature by very accurate controlling systems
- Heavy-duty, lockable and shock proof carrying case ensures safe transport
- Outer dimensions: 584 x 267 x 289 mm (L x W x H)
- Weight of complete kit: 11.5 kg

Embryo transport kit with incubator and battery 19180/2102




Transfit Lateral sheaths and ET Cannula

Sheath with lateral opening and metal tip, used with 0.25 ml straws for the transfer of bovine embryos.

Transfit Lateral [1]

sterilized, 5/bag	19240/1005
single packed in hygiene cover, 25/Ziplock bag	19240/1007

ET Cannula to be used with Transfit Lateral sheath [2] 19240/1000

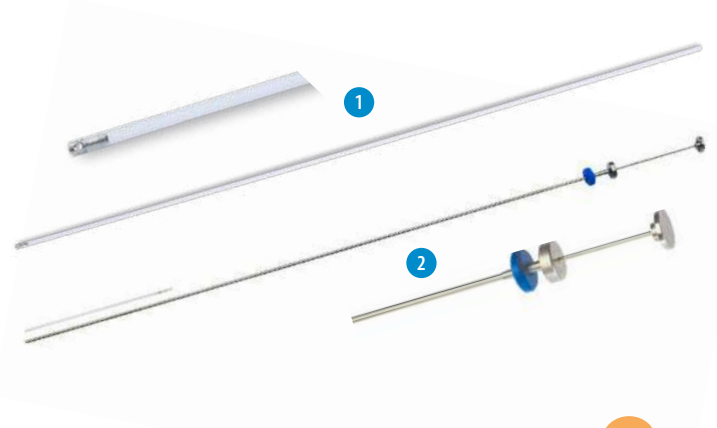

ET devices heater

Please see QuickLock Heater 4.0 on page 36.


Stainless steel ET instrument MT

- Best choice for heifers when cervix is hard to pass
- Sterilizable stainless steel device to be used without sheaths
- Detachable tip made of stainless steel with one side outlet
- Finger support ring with a groove marks the position of the outlet at the tip
- Disposable plastic tips are available

Transfer instrument, used with 0.25 ml straws [1] 19250/0000

Stainless steel tip [2] 19250/0989

Disposable tip, sterilized, single packed, 100/package [3] 19250/0991

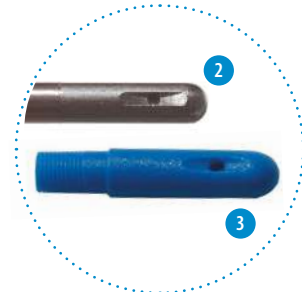

Hygienic sheaths

Used during AI and ET to keep the instruments clean while they pass the vagina and until they reach the cervix.

Hygienic sheaths

length: 46 cm (for AI), 80/box [1]	17028/0100
length: 53 cm (for ET), 80/box [2]	19271/0080

Sanisheath, rigid hygienic sheath with perforable cap, 25/package [3] 19270/0025



■ Ultrasound device GE Versana Active

- Portable, easy-to-use ultrasound system for linear and convex sector probes
- High quality documentation and data transfer system
- Display modes: B-Mode, M-Mode
- PW Color Doppler with automatic calculation
- Realtime Triplex-Mode
- Coded Harmonic Imaging, CrossXBeam, Speckle Reduction Imaging (SRI-HD) for exceptional image quality
- Connectivity: USB, HDMI, Ethernet
- Monitor: 15.6" high-resolution LCD
- Dimensions: 39 x 5.8 x 36.2 cm (W x H x D)
- Weight: 5 kg with battery
- Electrical connection: 100-240 VAC/50-60 Hz



Ultrasonic device Versana Active [23360/0000](#)

(↻ Accessories

Micro-convex probe, 4.2 - 10.0 MHz [23360/0002](#)

Endorectal linear probe, 3.5 - 10.0 MHz [23360/0003](#)

■ Ultrasound probe holder

The probe holder is available to suit GE, SonoSite, Aloka, BCF or ECM probe. The needle unit is included.

- Very lightweight and smooth probe holder provides easy handling
- Material warms up to body temperature very quickly
- Slim design enables oocyte aspiration in young animals when combined with small probe
- Optimized needle outlets for recommended needles (Ref. 23360/1200 and Ref. 23360/1250, page 48)

Ultrasound probe holder

GE [23400/0120](#)

SonoSite [23400/0041](#)

Aloka [23400/0027](#)

BCF [23400/0115](#)

ECM [23400/0023](#)

Esaote [23400/0126](#)

Cover for probe holder, 144/package [23360/1260](#)



Upon request we offer the device also adapted to other probes.

Aspiration pump for OPU

OPU pump for follicle aspiration and oocyte collection in cows and heifers with durable casing.

- Includes media warming block
- Pump is operated by foot switch
- Pressure level and hysteresis are programmable by the user
- Aspiration pressure adjustable up to -375 mmHg
- Warming temperature adjustable up to +50°C

Aspiration pump [1]	
230 V	23362/0000
115 V	23362/0001

Accessories

Stopper assembly, snugly connects to 50 ml tubes, 1/package [2]	23360/1100
Tubing set, sterilized, individually packaged [3]	23360/1000
Disposable needle, standard/long bevel, 100/package	
for cows (1.2 x 75 mm) [4]	23360/1200
for heifers (0.9 x 70 mm) [5]	23360/1250



BoviPlus OPU recovery medium

Complete solution, ready-to-use, containing BSA or PVA and Heparin to avoid blood coagulation during the OPU procedure.

BoviPlus OPU recovery medium, 500 ml	
with BSA	19982/1281
with PVA	19982/1283



BoviPlus washing medium

Ready-to-use medium with BSA for washing of OPU or abattoir derived bovine oocytes following collection.

BoviPlus Oocyte washing medium, 500 ml	19982/1200
--	------------



Media for in vitro production (IVP) of bovine embryos

- Natural additives promote a more natural and healthier environment for gametes and embryos
- Come as a dry powder with sterile water for reconstitution included in each presentation
- After reconstitution, the media are ready to use with no further additives required
- Powder form simplifies handling and reduces the risk of contamination
- Longer shelf life compared to liquid formulations, ensuring consistent performance over time
- Cost-effective and easy transport (can be shipped at room temperature)
- Each batch of media undergoes rigorous quality control, including Bovine Embryo Assay and sterility testing

NaturARTs Maturation Media

Enriched with natural additives to provide an even better environment for the oocytes.



Washing of oocytes and embryos

Wash Medium, 50 ml [1] 19990/0100

Maturation of oocytes

Maturation Medium, 10 ml [2] 19990/0110

Maturation Medium with HEPES, 10 ml 19990/0111

NaturARTs Maturation Medium, 10 ml [3] 19990/0113

NaturARTs Maturation Medium with HEPES, 10 ml 19990/0112

In vitro fertilization

Fertilization Medium, 10 ml [4] 19990/0140

Sperm preparation

Swim-up Medium, 20 ml [5] 19990/0120

Sperm Wash Medium, 20 ml [6] 19990/0130

In vitro culture

Culture Medium, 10 ml [7] 19990/0150

IVP media test kit

Contains: Wash Medium, NaturARTs Maturation Medium, Swim-up Medium, Sperm Wash Medium, Fertilization Medium, Culture Medium.

IVP media test kit [8] 19990/0099





MINITUBE INTERNATIONAL

Servicing reproduction technologies worldwide

Minitube USA

800.646.4882
info@minitube.com

Minitub do Brasil

+55 51 3222 7876
minitub@minitube.com.br

Minitube India

+91 11 4309 0668
info@minitube-india.com

Minitube China

+86 185 1545 5770
info@minitube.cn

Minitub Ibérica

+34 977 845 397
info@minitub-iberica.com

Minitube Australia

+61 3 5342 8688
info@minitube.com.au

Minitube Central Europe

+421 377 877 915
mtce@minitube.sk

Minitube Canada

800.500.7978
info.canada@minitube.com

Minitube México

+52 442 246 73 46/47
info@mexitube.mx

Minitube Middle East

+971 426 68871
info.middle.east@minitube.com

Minitube Indonesia

+62 21 7591 75-96/94
info@minitube.co.id

Minitube International AG, Germany

+49 8709 9229 0
minitube@minitube.de

Follow us



www.minitube.com

